



3M™/Solventum ActiV.A.C.™ Home Therapy Unit Instructions

1. Attach the power cord/power supply/charging cord to the therapy unit and plug into wall

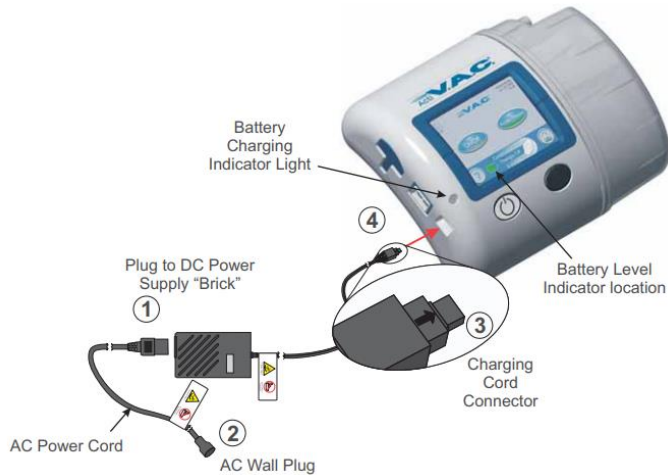

2. Attach the canister to the therapy unit and clamp the canister tubing

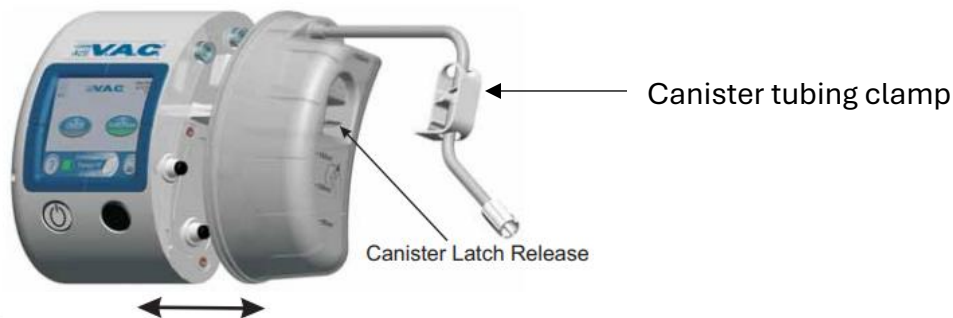

! *Canister is always applied straight on and straight off the ActiV.A.C.® Therapy Unit. Do not twist or turn canister when installing or removing.*


3. Turn the therapy unit on by pressing and holding the Power On/Off button for 2 seconds.




Power On/Off Button


- a. The therapy unit will perform a self-check
- b. Press "OK" on the home screen

- Clamp the dressing tubing AND the canister tubing on the hospital therapy unit

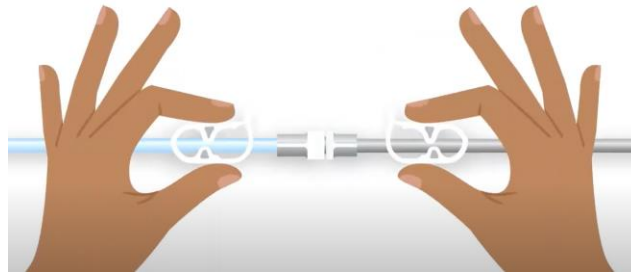

- Turn off the hospital therapy unit
- Disconnect the dressing tubing from the canister tubing on the hospital therapy unit
- Connect the dressing tubing to the canister tubing on the home therapy unit

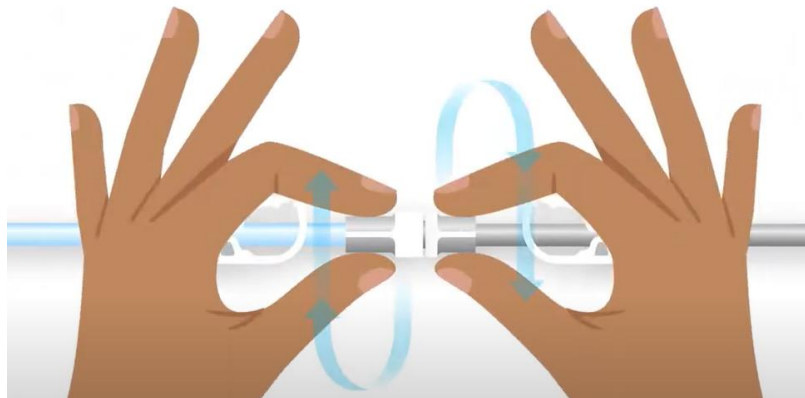

- Unclamp the canister tubing and the dressing tubing
- Press the on/off button on the therapy unit screen
- Confirm the dressing is compressed and the therapy icon  is spinning (indicating negative pressure is active.)

