


# RENASYS<sup>®</sup> GO

Negative Pressure Wound Therapy

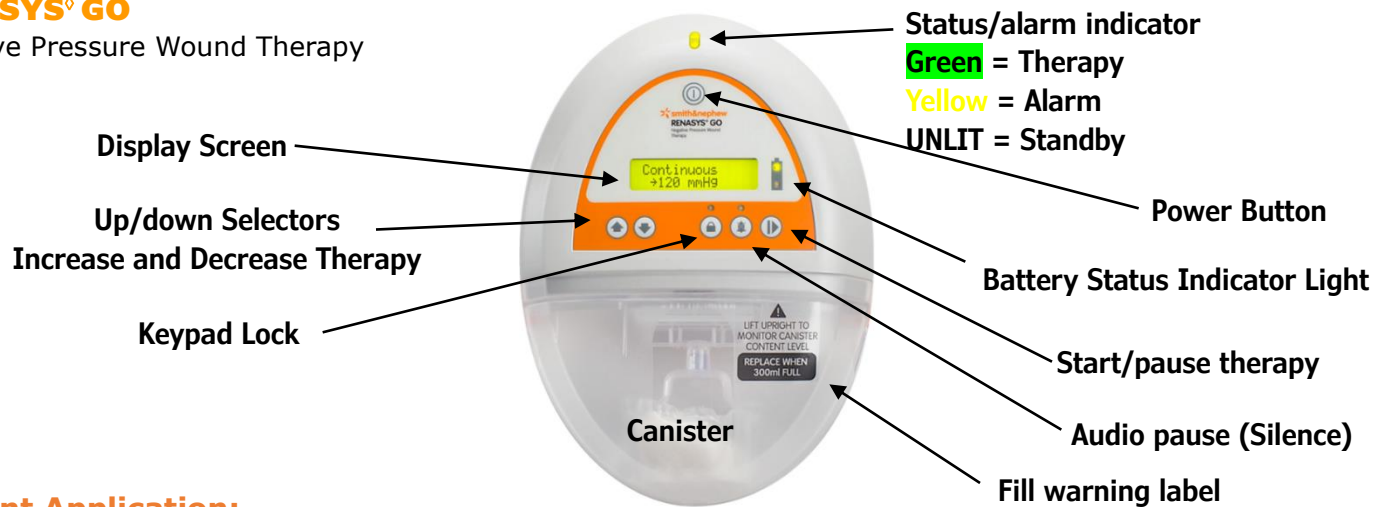

## Patient Application:

Step 1: connect new canister to Renasys Go device by opening both orange clips on the new canister. Push canister over inlet port until clips click in place.




### Attaching new canister:

- Ensure device is off
- Open both orange clips on the sides of canister
- Align canister so that viewing window is facing forward
- Push canister gently over device inlet port
- Engage both canister clips. Clips will click when properly engaged
- Resume therapy

Step 2: Turn off hospital negative pressure device.

Step 3: Disconnect canister tubing on hospital device from dressing tubing (do not remove dressing from patient).

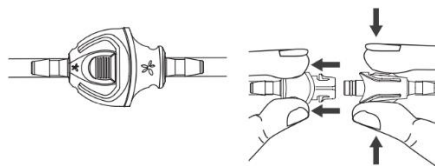

Step 4: Connect dressing tubing to Renasys Go home device tubing

Step 5: turn Renasys home device on and set pressure according to MD order

### Pressure Setting: 40 to 200mmHg: tolerance +/- 10 mmHg

- The device adjustments can be made in **10mmHg increments** between 40mmHg and 100mmHg
- **20mmHg increments** between 100mmHg and 200mmHg
- **Set per MD orders**
- **Continuous or Intermittent:**

For intermittent settings: Down + Select + Power buttons for 2 seconds.  
Intermittent mode = 5 minutes "ON" (active) and 2 minutes "OFF" (no vacuum)

Step 6: ensure dressing is compressed (indicating pressure is adequately applied)

For alarms or troubleshooting, please see home health user manual (posted separately)

RENASYS<sup>®</sup> GO Device Patient Instructional Video

RENASYS GO Quick Start Guide and Canister Change

